## **Rhino Care**

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## **NASAL WASH USER INSTRUCTIONS (RHINO CARE)**

**Rhino Care** is a medical device with risk class IIa intended to be connected to an active medical device in Class IIa or a higher class (e.g. Aerosol Therapy Equipment).

**Rhino Care** is a device used to clean the nasal cavities, hydrates the nasal mucous membranes and generates a micronised nasal spray solution to treat problems of the respiratory tract. It is recommended to use the spray for colds, dry nose, exposure to dust, pollen allergies, chronic inflammation, and when recommended by your doctor.

**The nasal wash must only be done using an isotonic or hypertonic saline solution.**

### **WARNINGS**

#### **GENERAL SAFETY REGULATIONS**

**Please read the instructions carefully before use. Keep the instructions in a safe place.**

- Do not use the nasal wash with drugs, balsamic oils and/or other medicinal solutions.
- Rhino Care does not work with ultrasonic aerosol devices.
- The device has small components which might be removed and easily swallowed. Use by minors and disabled people require presence of an adult with his

faculties. Don't leave the device unattended in places easily accessible by minors and disabled people.

- Store and use the device in places protected against the weather and far from any sources of heat. After each use, it is recommended to store the device in its own box away from dust and sunlight.
- **This medical device must be destined exclusively for the use for which it has been designed and described in this manual.** Any different use must be considered incorrect and therefore dangerous; the manufacturer cannot be considered liable for damage caused by improper, incorrect and / or unreasonable use or if the appliance is used in electrical plants that are not in compliance with the regulations in force;
- Do not leave the device unattended in places accessible to children and / or persons not in full possession of their mental;
- Dispose of device ads provided by standards in force.
- Disposal of the medical device must be performed according to specific laws in force in each country;
- Service life: 5 years from date of purchase. It is recommended to replace the device after long periods of inactivity or if it is deformed or broken.

## WARRANTY

- **Rhino Care** has a 2-year warranty from the date of purchase against any material or manufacturer defects;
- The warranty replaces/repairs the defective parts;
- The device must only be repaired by an authorised Safety spa technical service centre;
- The warranty shall be null and void if the device is tampered with or the defect is caused by improper use;
- The warranty is valid from the date of purchase of the product on confirmation of the tax receipt or purchase invoice.

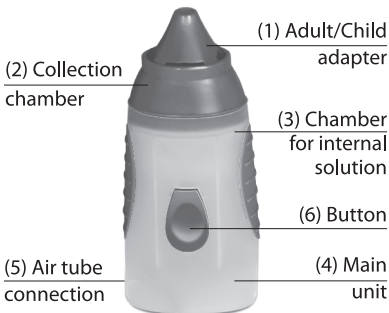
## TECHNICAL CHARACTERISTICS

Operating pressure range	90 – 140 kPa
Max Capacity	10 ml
Particle Size	40 $\mu$ m
Weight	Approx. 50gr
Dimension	14 x 6 x 4.8 cm
Working Condition	Room temperature: 5 ÷ 35 °C Room humidity percentage: 15 ÷ 93% RH Atmospheric Pressure: 700 ÷ 1060 hPa
Conservation condition and Transport	Room temperature: -25 ÷ 70°C Room humidity percentage: 0 ÷ 93% RH Atmospheric Pressure: 500 ÷ 1060 hPa

## SYMBOLOLOGY

 0123	CE marking in conformity with EC directive 93/42/EEC and subsequent changes
	Consult the instruction manual
	General warnings and/or specifications
	Manufacturer
	Lot number
	Identification device

## COMPOSITION OF THE DEVICE

## USE OF THE DEVICE

- Lift the adult or child adapter (1) and the collection chamber (2) situated on the main unit (4) of the nasal wash;

- Fill the chamber (3) with isotonic or hypertonic solution (max 10ml);
- Put the collection chamber and selected adapter back onto the top of the nasal wash, making sure the parts are perfectly in place;
- Connect the nasal wash (5) to the aerosol therapy device by means of an air tube (not supplied);
- Place the wash with the selected adapter under a nostril;
- Start the aerosol therapy device (refer to the user manual) and start the wash by pressing and holding the supply button (6);
- During treatment, breathe normally through your nose whilst keeping the other nostril closed;
- Alternate nostrils according to requirement
- When the liquid in the device finishes, or if you want to temporarily stop the treatment, release the nasal wash start button.
- After treatment, empty the collection chamber and clean it.

## **CLEANING THE DEVICE**

**Due to risk of contamination, empty the device each time it is used.**

Rhino Care nasal wash must be cleaned every time it is used. The components must be disassembled as follows:

1. Remove the air connection tube;
2. Remove the adapter (adult or child) and the collection chamber from the top of the nasal wash;

3. Remove the green impacter situated inside the nasal wash filling chamber.

The nasal wash top and collection chamber are a single piece and cannot be disassembled.

Once all components are disassembled, proceed with cleaning: Thoroughly clean the components for 5 minutes, using warm drinking tap water and/or mild soap.

After cleaning, rinse thoroughly under running water and removing excess water, and let it dry in a clean place.

**Do not use a microwave to clean or disinfect the device.**

## **MAINTENANCE**

Cleaning the components of the nasal wash enable you to constantly check their appearance and make it easier to see when they need to be replaced.

Rhino Care nasal wash must be replaced when signs of deterioration are visible.

Rhino Care has no parts that require maintenance. However, it is necessary to perform a few simple checks to verify its functionality before each use.

## **Problems**

No nebulised solution comes out from the Rhino Care nasal wash

### **Possible Causes**

The aerosol therapy device is off

### **Solution**

Switch on the aerosol therapy device

---

### **Possible Causes**

No saline solution in the chamber

### **Solution**

Pour saline solution into the chamber (max 10ml)

---

### **Possible Causes**

Rhino Care start button not pressed

### **Solution**

Press and hold the button

---

### **Possible Causes**

Air tube not connected correctly or the nasal wash was incorrectly assembled

### **Solution**

Connect the air tube correctly to the nasal wash connection and air supply connection of the aerosol therapy device.

Make sure that all parts of the nasal wash are assembled correctly

---

If Rhino Care still does not work after the aforesaid conditions have been checked, it is recommended to contact your dealer or Safety S.p.A. technical service.

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Distribuito da: Safety S.p.A.