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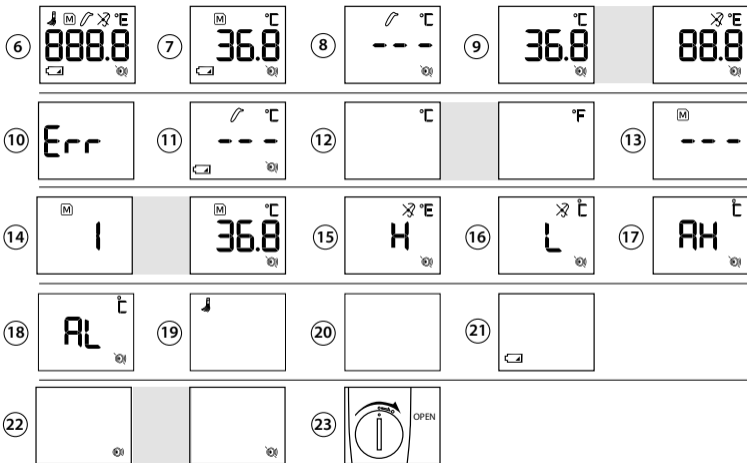
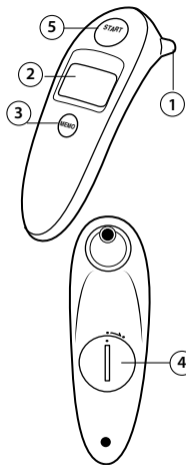
EAR THERM

Termometro auricolare
Ear thermometer

MANUALE D'USO
INSTRUCTION OF USE

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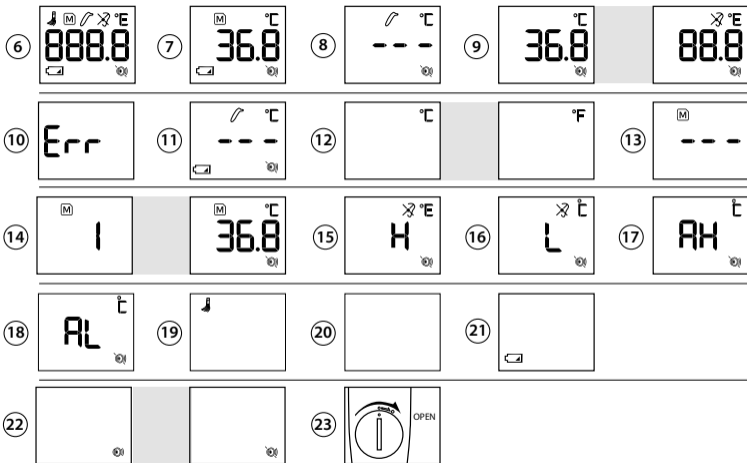
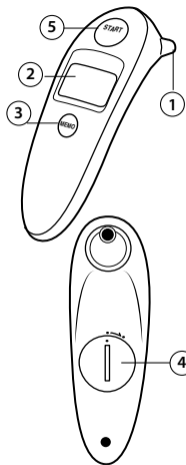
- 1 Measuring sensor
- 2 Display
- 3 Memo button
- 4 Battery compartment cover
- 5 START/IO button
- 6 All segments displayed
- 7 Memory
- 8 Ready for measuring
- 9 Measurement complete
- 10 Error function display
- 11 Low battery indicator

- 12 Changing between Celsius and Fahrenheit
- 13 Recall mode
- 14 Recall the last 10 readings
- 15 Measured temperature too high
- 16 Measured temperature too low
- 17 Ambient temperature too high
- 18 Ambient temperature too low
- 19 Clean reminder
- 20 Blank display
- 21 Flat battery
- 22 Beeper function setting
- 23 Replacing the battery

Read the instructions carefully before using this device

Type BF applied part

EAR THERM, ear thermometer is a high quality product incorporating the latest technology and tested in accordance with international standards. With its unique technology, this device can provide a stable, heat-interference-free reading with each measurement. The device performs a self-test every time it is switched on to always guarantee the specified accuracy of any measurement. **EAR THERM**, ear thermometer is intended for the periodic measurement and monitoring of human body temperature. It is intended for use on people of all ages.

This thermometer has been clinically tested and proven to be safe and accurate when used in accordance to the operating instruction manual. Please read through these instructions carefully in order for you to understand all functions and safety information.

Table of Contents

- 1 The Advantages of this Thermometer
 - Measurement in 1 second
 - Multiple uses (wide range of measurement)
 - Accurate and reliable
 - Gentle and easy to use
 - Multiple readings recall
 - Safe and hygienic
 - Fever alarm
- 2 Important Safety Instructions
- 3 **How this Thermometer measures Temperature**
 - To avoid an inaccurate measurement
- 4 Control Displays and Symbols
- 5 Setting Beeper Functions
- 6 Directions for Use
- 7 Changing between Celsius and Fahrenheit

- 8 How to recall 10 readings in Memory Mode
- 9 Error Messages
- 10 Cleaning and Disinfecting
- 11 How Product calibration, maintenance and maintenance methods
- 12 Battery Replacement
- 13 Guarantee
- 14 Technical Specifications

1. The Advantages of this Thermometer

Measurement in 1 second

The innovative infrared technology allows measurement of ear temperature in only 1 second.

Multiple uses (wide range of measurement)

This thermometer offers a wide range of measurement from 0.1 - 99.9 °C / 32.2 - 211.8°F, meaning the unit

can be used to measure body temperature or it also has a feature allowing it to be used to measure surface temperature of the following examples:

- Milk surface temperature in a baby's bottle
- Surface temperature of a baby's bath
- Ambient temperature

Accurate and reliable

The unique probe assembly construction incorporates an advanced infrared sensor, ensuring that each measurement is accurate and reliable.

Gentle and easy to use

- The ergonomic design enables simple and easy use of the thermometer.
- This thermometer is quick, therefore child-friendly.

Multiple readings recall

Users will be able to recall the last 10 readings when entering the recall mode, enabling efficient tracking of temperature variations.

Safe and hygienic

- No risk of broken glass or mercury ingestion.
- Completely safe for use on children.
- Cleaning of the probe can be done with an alcohol-moistened cotton tissue, making this thermometer completely hygienic for use by the whole family.

Fever alarm

10 short beeps alert the patient that he/ she may have a temperature equal to or higher than 37.5 °C.

2. Important Safety Instructions

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- **Never immerse this device in water or other liquids. For cleaning please follow the instructions in the «Cleaning and Disinfecting» section.**
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- Earwax in the ear canal may cause a lower temperature reading. Therefore it is important to ensure the

subject's ear canal is clean.

- If the measurement result is not consistent with the patient's finding or unusually low, repeat the measurement every 15 minutes or double check the result by another core body temperature measurement.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

- Protect it from:
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold

WARNING: The measurement results given by this device is not a diagnosis. It is not replacing the need for the consultation of a physician, especially if not matching the patient's symptoms. Do not rely on the measurement result only, always consider other potentially occurring symptoms and the patient's feedback. Calling a doctor or an ambulance is advised if needed.

3. How this Thermometer measures Temperature

This thermometer measures infrared energy radiated from the eardrum and the surrounding tissue. This energy is collected through the lens and converted to a temperature value. The measured reading obtained directly from the eardrum (Tympanic Membrane) ensures the most accurate ear temperature.

Measurements taken from the surrounding tissue of the ear canal generate lower readings and may result in misdiagnosis of fever.

To avoid an inaccurate measurement

- Switch on the thermometer by pressing the START/IO button (5).
- After one beep is heard (and the ready for measurement icon show), straighten the ear canal by gently pulling the middle of the ear back and up.

- Press the START/IO button (5). Release the button and wait for the beep sound. This indicates the end of the measurement.
- Place the probe (1) firmly into the ear canal. Press the START/IO button (5) and keep the probe in the ear until the thermometer beeps to signal the completion of the measurement.

4. Control Displays and Symbols

- **All segments displayed (6):** Press the START/IO button (5) to turn on the unit; all segments will be shown for 1 second.
- **Ready for measuring (8):** When the unit is ready for measurement, the «°C» or «°F» icon will keep flashing.

- **Measurement complete (9):** The reading will be shown on the display (2) with the «°C» or «°F» icon; the unit is ready for the next measurement, when the «°C» or «°F» icon is flashing again (and the ready for measurement icon show).
- **Low battery indicator:** When the unit is turned on, the «battery» icon will keep flashing to remind the user to replace the battery.

5. Setting Beeper Functions

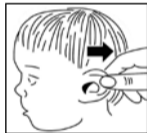
Setting the beeper

- When the device is off, press and hold the START/IO button (5) for 5 sec to set the beeper
- Press the START/IO button (5) to either turn the beeper on or off. The beeper is activated when the beeper icon is shown without a cross.

6. Directions for Use

- Press the START/IO button (5). The display (2) is activated to show all segments for 1 second.
- When the «°C» or «°F» icon is flashing, a beep sound is heard and the thermometer is ready for measuring (8).
- Straighten the ear canal by pulling the ear up and back to give a clear view of the eardrum.

- For children under 1 year;
pull the ear straight back.

- For children of 1 year to adult;
pull the ear up and back.

- While gently pulling the ear, insert the device snugly into the ear canal.
- Press the START/IO button (5). Release the button and wait for the beep sound. This indicates the end of the measurement.
- Remove the thermometer from the ear canal. The display shows the measured temperature (9) .
- Clean the device before starting a new measurement.
- For the next measurement, wait until the «°C»/«°F» icon is flashing and follow steps 3-5 above.
- Press and hold the START/IO button for 3 seconds to turn off the device; otherwise the device will automatically switch off after approx. 60 seconds.
- The broom (19) will be shown to remind user of cleaning the device before turning off the device

NOTE

Patients and thermometer should stay in similar room condition for at least 30 minutes.

- **To ensure accurate readings, wait at least 30 sec. after 3-5 continuous measurements.**
- **After cleaning the measuring sensor (1) with alcohol, wait 5 minutes before taking the next measurement,** in order to allow the thermometer to reach its operating reference temperature.
- 10 short beeps alert the patient that he/ she may have a temperature equal to or higher than 37.5 °C.
- For an infant, it is best to have the child lying flat with his/her head sideways so the ear is facing upwards. For an older child or adult, it is best to stand behind and slightly to the side of the patient.
- Always take the temperature in the same ear, since the

temperature readings may be different from ear to ear.

- Don't take a measurement while or immediately after nursing a baby.
- Don't use the thermometer in high humidity environments.
- Patients should not drink, eat or exercise before/ while taking the measurement.
- Doctors recommend rectal measurement for newborn infants within the first 6 months, as all other measuring methods might lead to ambiguous results.
- **Readings from different measuring sites should not be compared as the normal body temperature varies by measuring site and time of day**, being highest in the evening and lowest about one hour before waking up.

Normal body temperature ranges:

- Axillar: 34.7 - 37.3 °C / 94.5 - 99.1 °F
- Oral: 35.5 - 37.5 °C / 95.9 - 99.5 °F
- Rectal: 36.6 - 38.0 °C / 97.9 - 100.4 °F

7. Changing between Celsius and Fahrenheit

This thermometer can display temperature readings in either Fahrenheit or Celsius. To switch the display between °C and °F, when the power is off, press and hold the Memo button (3) for 5 seconds; The current measurement scale («°C» or «°F» icon) will be shown on the display (2) and flash. Change the measurement scale between °C and °F by pressing the Memo button (3) again. When the measurement scale has been chosen and no button is pressed in 5 seconds, the device enters automatically the ready for measurement mode.

8. How to recall 10 readings in Memory Mode

This thermometer can recall the last 10 readings

- **Recall mode 13:** Press the Memo button (3) to enter recall mode .
- **1st to 10th reading (14):** Press and released the Memo button (3), a number «1» will show for 1 second on monitor then follow by the «1st» reading with icon «M» flashing .Press and release the Memo button (3) again to get following readings sequentially. After the last 10 readings have been recalled, the device will resume the above sequence from reading 1.

9. Error Messages

- **Measured temperature too high 15:** Displays «H» when measured temperature is higher than 99.9 °C / 211.8 °F.

- **Measured temperature too low 16:** Displays «L» when measured temperature is lower than 0.1 °C/32.2 °F
- **Ambient temperature too high 17:** Displays «AH» when ambient temperature is higher than 40.0 °C/104.0 °F.
- **Ambient temperature too low 18:** Display «AL» when ambient temperature is lower than 10.0 °C / 50.0 °F.
- **Error function display 10:**
The system has a malfunction.
- **Blank display 20:** Check if the battery has been loaded correctly. Also check polarity (<+> and <->) of the battery.
- **Flat battery indicator:** If only the «battery» icon is shown on the display, the battery should be replaced immediately.

10. Cleaning and Disinfecting

Use an alcohol swab or cotton tissue moistened with alcohol (70% Isopropyl) to clean the thermometer casing and the measuring sensor. Ensure that no liquid enters the interior of the device. Never use abrasive cleaning agents, thinners or benzene for cleaning and **never immerse the device in water or other cleaning liquids**. Take care not to scratch the surface of the sensor lens and the display.

11. How Product calibration, maintenance and maintenance methods

- The thermometer does not need to be calibrated within the 2 year. If correction is required outside the guarantee, Please refer to the “Guarantee” and contact your dealer.

- After each use, the product should be cleaned according to the instructions of “cleaning and disinfection”, and the product will be stored in the appropriate humidity temperature and conditions of the storage environment.
- For a long time remove the battery from the product when not in use.

12. Battery Replacement

This device is supplied with one lithium battery, type CR2032. The battery needs replacing when only the «battery» icon is shown on the display.

Remove the battery cover by sliding it in the direction shown. Insert the new battery with the + at the top. Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

13. Guarantee

This device is covered by a **5 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

- The guarantee covers the device. The battery and packaging.

Are not included.

- Opening or altering the device invalidates the guarantee.
- The guarantee does not cover damage caused by improper handling, a discharged battery, accidents or non-compliance with the operating instructions. Please contact Safety-service.

14. Technical Specifications

Type: Ear Thermometer EAR THERM

Measurement: 0.1 – 99.9 °C/ 32.2 – 211.8 °F

Resolution: 0.1 °C / °F

Measurement accuracy (Laboratory):

Body mode:

±0.2 °C, 35.0 ~ 42.0 °C / ±0.4 °F, 95.0 ~ 107.6 °F

±0.3 °C, 32.0 ~ 34.9 °C and 42.1 ~ 43.0 °C /

±0.5 °F, 89.6 ~ 94.8 °F and 107.8 ~ 109.4 °F

Object mode:

±1.0 °C, 0.1 ~ 31.9 °C, 43.1 ~ 99.9 °C /

32.2 ~ 89.4 °F, 109.6 ~ 211.8 °F

Display: Liquid Crystal Display, 4 digits plus special icons

Acoustic: The unit is turned ON and ready for the measurement: 1 short beep
Complete the measurement: 1 long beeps

System error or malfunction: 3 short beeps
Fever Alarm: 10 short beeps

Memory: 10 readings recall in the memory mode

Operating conditions:

10 - 40 °C / 50.0 - 104 °F (70kpa~106kpa)

15 - 95 % relative maximum humidity

Storage conditions:

-25 - +55 °C / -13 - +131 °F

15 - 95 % relative maximum humidity

Automatic Switch-off:

Approx. 1 minute after last measurement has been taken.

Battery: 1 x CR2032 battery 3V

Battery lifetime:

Approx. 2000 measurements (using a new battery)

Dimensions: 127 x 40 x 61 mm

Weight: 58 g (with battery), 55 g(w/o battery)

IP Class: IP22

Reference to standards:

ASTM E1965; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11

Expected service life: 5 years or 12000 measurements

This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

According to the Medical Product User Act a biennial technical inspection is recommended for professional users.

Please observe the applicable disposal regulations

 Microlife Corporation 9F, 431, RuiGuang Road, Nei-Hu. Taipei 114. Taiwan, ROC

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 22890

rev01_mar2021

The logo for Safety S.p.A. features a stylized graphic of three vertical lines of varying heights on the left, followed by the word 'SAFETY' in a bold, uppercase, sans-serif font.

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Distributore: Safety S.p.A. Via G. Di Vittorio, 17 - 20813 Bovisio Masciago (MB) Italy